Luce D40 • Luce D60 • Luce D90 • LED Vision

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# The importance of lighting for the dentist

The lighting system of the operating field in the surgery is a priority aspect. Many professionals like dentists, dermatologists, gynecologists put into effect activity of diagnosis, inspection and surgery. A good and adapted lighting system allow one better operativity and one visual protection.

In case of the dentist requirement it is still marked because he works in a dark and not linear cavity.

## The lighting surgery

Surgical techniques of dentists have become more sophisticated and demanding, hence dentists work must not be affected by any element. The intervention by more than one person and the position of the dentist's body creates shadows affecting the dentists work. The new single lamp surgery lights by Tecno-Gaz eliminate these problems and ensure a perfect lighting also in case of parallel work.

## Advantage of the surgery lamps by Tecno-Gaz:

### PERFECT AND WIDE LIGHTING OF THE OPERATIVE AREA

- Advantages for the dentists sight and improved visibility with regard to current lighting methods
- Greater ability in focusing details
- Distinction between venous and arterial blood
- Easier identification of colours

### HEAT COOLING

### FOCAL ALWAYS PERFECT

- Thus avoiding the continuous adjustment of the light-patient distance

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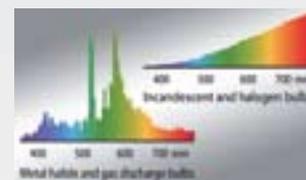
## Special features of surgical lamps

### Faithful colour reproduction

The reflectors of our lamps are treated with a special multilayer "SUN LIGHT" coating which enhances the reflection and ensures a colour rendering index equivalent to 94ra. A fantastic technical achievement which guarantees operators a colour rendering equivalent to that of natural daylight, even in the most extreme conditions.

### Depth of light

To provide maximum depth, Tecno-Gaz lamps have been especially designed to use reflectors with geometry composed of dozens of interconnected ellipsoids which direct the rays to form a cone of light with the power to penetrate to a considerable depth.

### No shadows

The structures of the elliptical parabola are especially designed to reflect thousands of pinpoint rays of light indirectly on the surgical site, thereby eliminating all areas of shadow.

### Cold light

Tecno-Gaz has dedicated particular attention to this aspect, adopting special "RMS97 filters" which absorb and eliminate the infrared rays generated by the halogen bulb, guaranteeing a cold light.

### Colour temperature

The "sun light" coating applied to the ellipse and the "dichroic" coating applied to the parabola raise the colour temperature to 4,200°K, generating a white light similar to that of the sun, thereby protecting the dental surgeon's sight and guaranteeing absolute precision of vision.

### Visible wavelength

Tecno-Gaz lamps use halogen bulbs, because the visible spectrum emission is most similar to that of sun light and this guarantees visual consistency.

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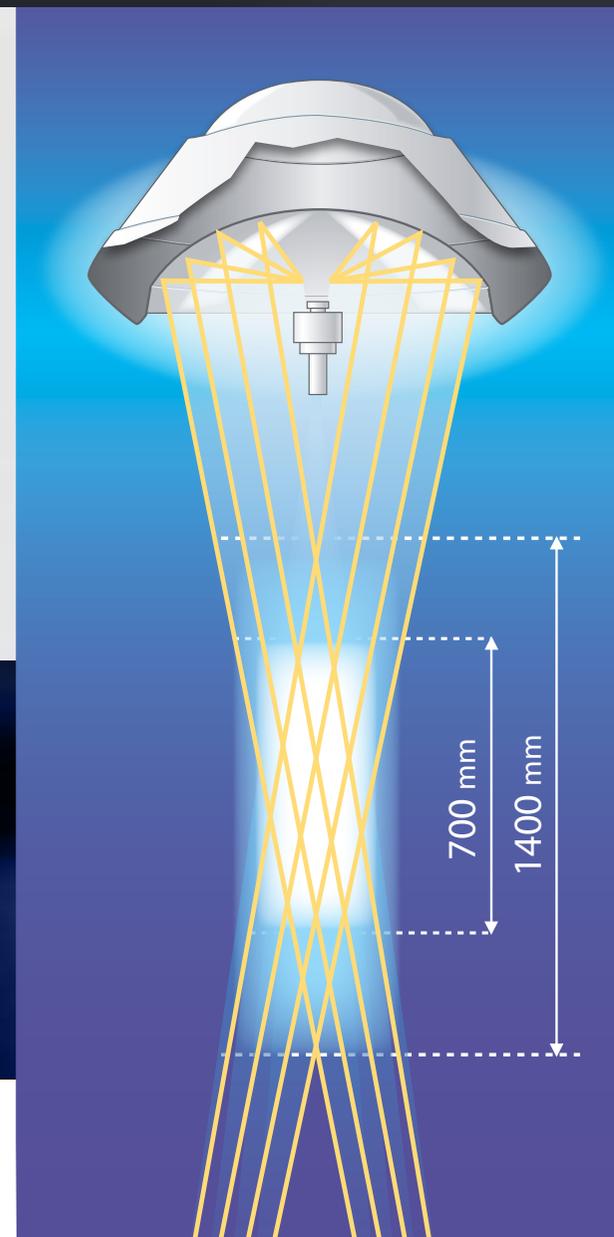
## An innovative system "ELLIPTIC CONCEPT"

"The elliptic concept" is a patented system that will characterize the future of new generation surgery lamps.

**This concept derives from the combination of physical studies and mathematic calculations and belongs exclusively to the surgery lamps of luce.**

The parabola is built with a special shape and then submitted to stratified chromium plating treatment. This allows a strong increase of illumination and the use of a low voltage halogen lamp. Such solution offers a perfect lighting and a strong heat reduction.

- Heat reduction thanks to the presence of special filters eliminating 98% of IR rays
- Anti-reflection glass that grants indirect light
- Focal with a wide adjusting spectrum
- Two-lamp electronic system with automatic switch
- Flashing indicator warning of the non functioning lamp
- Light beam adjustable through autoclavable support

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## Special features of the LUCE line lamps

- Tecno-Gaz lamps are always focused at a minimum distance of 60 cm.
- The diameter of the illuminated field may be adjusted manually by turning the knob located at the centre of the reflector. This knob may be easily removed by the operators for sterilization purposes.
- All Tecno-Gaz models are equipped with the lighting intensity control as standard, enabling electronic adjustment via potentiometer from 20.000 Klux up to maximum intensity. An absolutely indispensable option providing the dentist with a multi-purpose lamp suitable for all types of operating activities.
- Tecno-Gaz has dedicated particular attention to the aspects of manoeuvrability and ergonomics during operation. Surgery in an outpatients' context, especially dental surgery, requires a great many adjustments of the arm and, in order to guarantee ease of use, maximum manoeuvrability has been achieved in a structure made entirely of aluminium, which also has the advantage of being extremely light.
- The joints of the articulated arms have bushings to facilitate positioning and guarantee absolute stability.
- Thanks to its lightness, the device is extremely easy to mount on any type of ceiling.

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#### Automatic bulb changeover system \*

The lamps are equipped with a block composed of 2 halogen bulbs which are electronically controlled in the event of a failure in the main bulb. The second bulb automatically comes into operation in place of the faulty one. Simultaneously, a LED located on the cover of the lamp starts flashing indicating that the main bulb requires replacing.

#### Lamp replacement signal\*

There is a LED indicator on the cover of the lamp that starts flashing to attract the operator's attention so that he/she proceeds to replace the faulty bulb.

\* standard supply on models D60 and D90, on request on models D40.

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## Depth

The rays generated by the lamp are reflected by the elliptical reflector onto the surgical site in pinpoints to ensure that the light is constantly focused and three-dimensional. Thanks to this principle, the light is always focused at a minimum distance of 60 cm from the reflector.

## Spotlight control

The spotlight can be adjusted manually by turning the knob located at the centre of the reflector. This knob may be easily removed by the operators for sterilization purposes.

## Cold light

Special “RMS97 filters” eliminate the infrared rays generated by the halogen bulb, in order to safeguard against dehydration of tissue during surgery and facilitate the dental surgeon's work.

## Elliptical reflector

This is made of aluminium and is composed of dozens of interconnected ellipsoids. The elliptical shape guarantees excellent lighting intensity with a low voltage halogen bulb and the reflected light is always strongly focused. The elliptical reflector has a “mirror” coating to increase the reflection and a “sun light” coating to raise the colour temperature to 4.200 Klux with colour rendering index of 94.

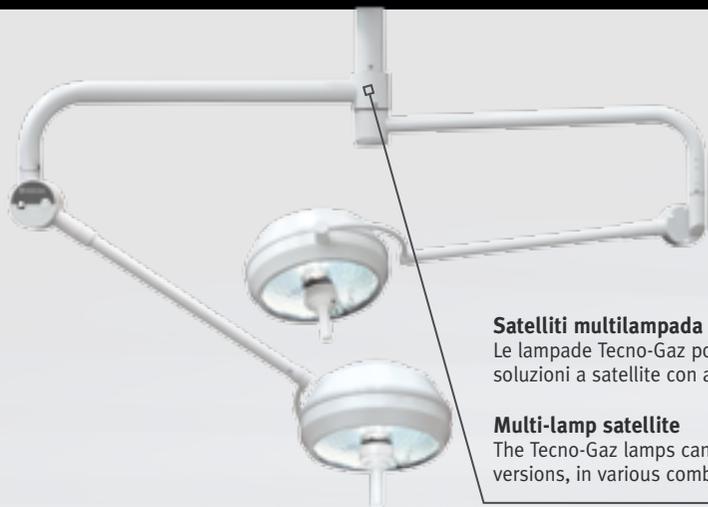
## Aluminium structure

The “lightstruc” load-bearing structure of the lamp is made of tubular aluminium with a diameter of 45 mm. This feature facilitates mounting, which can be carried out without the use of mechanical lifting equipment, and makes the product suitable for all types of ceilings as well as being extremely easy to manoeuvre. It also has excellent heat dissipation properties.

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### Multi-lamp satellite

The Tecno-Gaz lamps can be supplied in satellite versions, in various combinations on request.

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### Operating adjustments

The vertical arm has a yaw of 97° and a stroke of 1455 mm. It is balanced by a compression spring and can be regulated should this be required over time. It is light and its special bushings guarantee stability in all positions.

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### 10° Rotation

The standing version has been especially designed to be rotated by 10° on a vertical pole, without rotating the base. This technical solution makes it particularly suitable in surgeries where, for logistical reasons, it is difficult to move the base on wheels.

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### Anti-static wheels with brake

The mobile standing version is equipped with four anti-static wheels, two of which with brakes, as standard.

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**Lighting intensity control**

Lighting intensity can be electronically regulated thanks to a potentiometer from 20.000 Klux to maximum intensity.  
STANDARD SUPPLY.

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**Reserve bulb**

The lamp can be equipped with a reserve bulb. In the event of a failure in the main bulb, the device instantly switches on the reserve bulb.  
ON REQUEST.

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**Emergency battery**

Each Tecno-Gaz lamp can be equipped with an emergency battery, battery charger and electrical circuit.  
ON REQUEST.

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- Stativo | On trolley
- Parete | Wall mounted
- A soffitto | Ceiling

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### Lighting intensity control

Lighting intensity can be electronically regulated thanks to a potentiometer from 20.000 Klux to maximum intensity. The surgeon can set the lighting intensity required for the type of activity to be carried out.

### Operating adjustments

The vertical arm has a yaw of 97° and a stroke of 1455 mm. It is balanced by a compression spring and can be regulated, should this be required over time. It is light and its special bushings guarantee stability in all positions.

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### Lighting intensity control

Lighting intensity can be electronically regulated thanks to a potentiometer from 20.000 Klux to maximum intensity. STANDARD SUPPLY.

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### Reserve bulb

The lamp can be equipped with a reserve bulb. In the event of a failure in the main bulb, the device instantly switches on the reserve bulb. STANDARD SUPPLY.

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### Emergency battery

Each Tecno-Gaz lamp can be equipped with an emergency battery, battery charger and electrical circuit. ON REQUEST.

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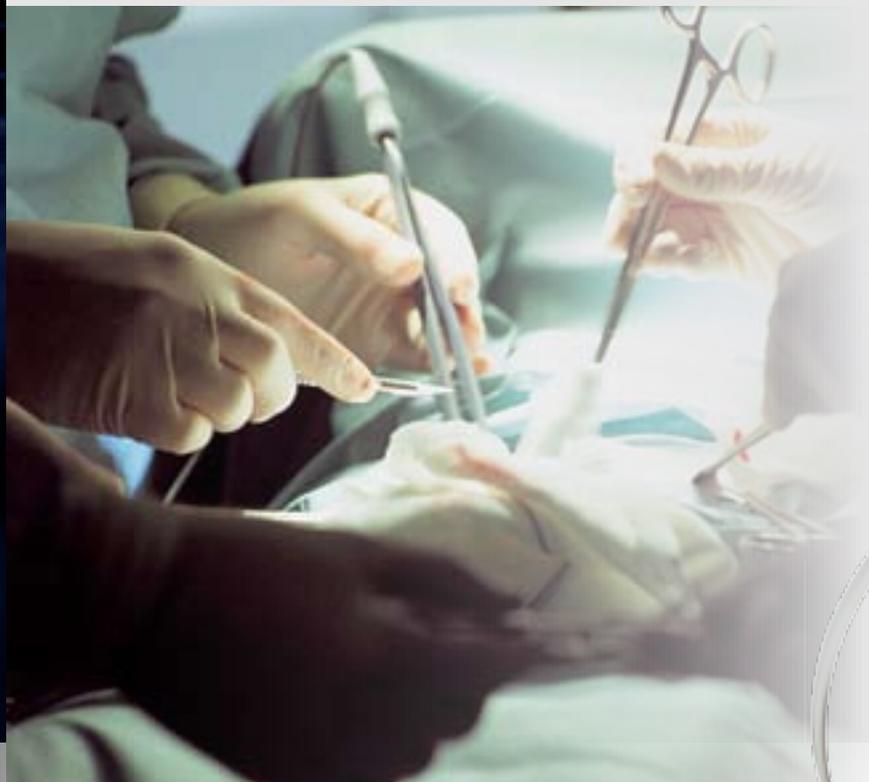
### Surgical mono-bulb lamp available in ceiling version

Equipped with automatic dual-bulb system with automatic changeover in the event of a failure in the main bulb and intermittent LED indicator on the cover. Spot regulation and lighting intensity control supplied as standard.

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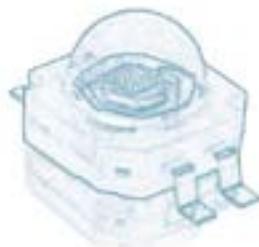
### In clinic, medicine and surgeon field

The importance of light during surgeries is related to the need to guarantee the reliability of the visual information perceived by the surgeon during the operation. What emerges clearly from luminous-technical studies is that "hot" lights (with longer wave lengths) excite more than "cold" ones (with shorter wave lengths). The IEC 60601-2-41 international regulation for surgery lamps envisages that the colour temperature should be between 3,000°K and 6,700° Kelvin. Studies on the physiological and psychological effects of the colour of light confirm the validity of the choice, as shown in the table.

#### Physiological and psychological effects on man

Effets	Light	
	Hot	Cold
<ul style="list-style-type: none"> <li>• Blood pressure</li> <li>• Breathing</li> <li>• Heart beat are</li> </ul>	Stimulated	Calmed
Eyelid movement	More frequent	Less frequent
The nervous system is	Stimulated	Relaxed
Acoustic sensitivity is	Minor	Major
Volumes appear	Reduced	

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## An innovative technology

The phenomenon of electro-luminescence, which is at the base of the functioning of LED devices, is caused by the generation of a radiation as a result of electricity passing through a body subjected to the effect of an electrical field. The spectral distribution of the radiation emitted, contains only a limited band of waves length. Thanks to IRC treatment, Tecno-Gaz is able to optimise the plurality of the colours of the visible spectrum.

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- 1 **Leds do not emit infrared rays so that the light generated is completely cold** "Cold" light is the fundamental prerogative of an excellent scialytic lamp, avoiding dehydration of the skin and tissues during the surgery and making the surgeon's work more comfortable.
- 2 **LEDs do not emit ultraviolet rays.** It has no dangerous repercussions for the skin and tissues.
- 3 **The life of LEDs is over 50,000 hours:** over twenty-five times that of the traditional halogen lamp LEDs never burn out so nobody is never left in the dark.
- 4 **It is a reliable and safe product over a life time, thanks to thermal and electrical testing.** Every smallest detail is studied, designed and tested by our Research Centre.
- 5 **The colour temperature always remains constant.** The colour temperature always remains constant regardless of the variation in luminous intensity.
- 6 **The lamp is designed to obtain a three-dimensionally.** Light the elliptical geometry of the parabola projects the luminous rays into a cone, illuminating deeply. Deep light illuminates human cavities three-dimensionally, helping to make even the smallest detail clear and facilitating
- 7 **Every parabola reflects homogeneous white light without showing edges of colour on the illuminated Part** - The sophisticated treatment, allows a faithful reproduction of the object colours without modifying their tints.
- 8 **Low consumption of power for a more economic solution.** Leds' extremely reduced consumption allows increasing the autonomy of the traditional set of batteries, in some cases, up to 8 hours of continuous use.
- 9 **Drastic reduction in maintenance costs.** LEDs last for over 50,000 hours and do not burn out. This means that the average life of a led is 25 times greater than that of an incandescent lamp.
- 10 **Ecological at 360° and simple to dispose off.** Unlike the sources of traditional light, Leds do not contain mercury, toxic gases, filaments or fragile parts.

# Led Vision L40

**LED VISION L40**  
Soffitto / Ceiling  
Art. 364-S

**LED VISION L40**  
Parete / Wall Mounted  
Versione a satellite  
a richiesta

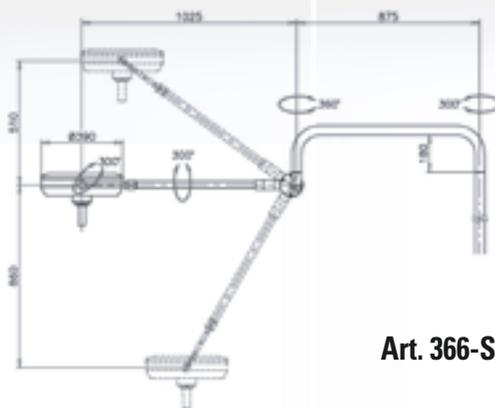
*Upon request we could  
also provide a dupple  
head (satellite version)*

**LED VISION L40**  
Stativo / On Trolley  
Art. 360-S

**LED  
VISION  
L40**

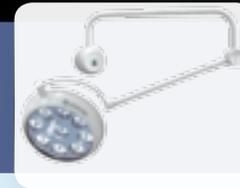
**LED VISION L40**  
Parete / Wall Mounted  
Art. 362-S

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## The best relationship for a better wellness

**Light affects the operative cycles of human beings in a fundamental way.** It has, in fact, been demonstrated that proper illumination improves comfort and reduces stress level; it has a positive direct effect on mood because it enhances the production of serotonin, thus reducing body fatigue and increasing human productivity. Illumination is the proper management of natural or artificial light sources aimed at obtaining a good vision. The importance of good artificial illumination is directly proportional to the difficulty of the task requested. Illumination must make carrying out a task easier: a good level of illumination stimulates good productivity and in particular it decreases errors, and it is less tiring for the eyes. **A good light must never glare, it must be well spread out and of overall good quality.** This is why, other than the average level of illumination needed, the presence of good quality light emitted from the sources, the uniformity of lighting and balanced luminescence must be taken into consideration, above all to avoid the phenomenon of glare. The colour yield of the colours of artificial luminous sources is extremely important. **Many artificial luminous sources emit light that is only apparently equal, above all if they have the same shade.** However, when lighting a coloured object, the tone and coloration will appear different, depending on the type of lamp illuminating it. This is due to the different "spectrum" of various lights, even if they are apparently equal. This different spectrum becomes evident when decomposing the light, this can be seen by letting a beam of light pass through a prismatic crystal (an experience well known to pupils at school). Even if the luminous source used in lighting interiors emits practically all the colours of the "rainbow" (the spectrum), different colours with different power are emitted, depending upon the type of lamp (incandescent, fluorescent in various tonalities, sodium, mercury, halogen).

# Lampade scialitiche | Lampade chirurgiche a LED Surgical lamps | Surgical LED lights

Strada Cavalli, 4 • 43038 Sala Baganza (PR) Italy  
Tel./Ph +39 0521 8380 • Fax +39 0521 833391  
info@tecnogaz.com

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All rights reserved. Variations can be done without notice. Tecno-Gaz S.p.A. is to be considered not responsible for damages caused by the lack or the wrongness of the information here mentioned.

[www.tecnogaz.com](http://www.tecnogaz.com)