SL

Lampade scialitiche  
Surgical lamps

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## The importance of lighting for the dentist

Dentist, contrarily to other physicians, carry out mainly a surgical work, hence they have to protect their hands and eyes, like other surgeons. that is why lighting is very important since the dentist operates in dark non linear hollows.

Futhermore, if you consider the high surgical level reached in many dentist's rooms the lighting of the operating field is of utmost importance. Consequently, dentists have to adopt new lighting system suitable to every changing requirements and to the protection of their sight.

## New requirements in dentistry

Surgical techniques of dentists have become more sophisticated and demanding, hence dentists work must not be affected by any element. The intervention by more than one person and the position of the dentist's body creates shadows affecting the denists work.

The new single lamp surgery lights by Tecno-Gaz eliminate these problems and ensure a perfect lighting also in case of parallel work.

### Advantage of the surgery lamps by Tecno-Gaz:

#### PERFECT AND WIDE LIGHTING OF THE OPERATIVE AREA

- Advantages for the dentists sight and improved visibility with regard to current lighting methods
- Greater ability in focusing details
- Distinction between venous and arterial blood
- Easier identification of colours

#### HEAT COOLING

#### FOCAL ALWAYS PERFECT

- Thus avoiding the continuous adjustment of the light-patient distance

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# Special features of surgical lamps

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The reflectors of our lamps are treated with a special multilayer "SUN LIGHT" coating which enhances the reflection and ensures a colour rendering index equivalent to 94ra. A fantastic technical achievement which guarantees operators a colour rendering equivalent to that of natural daylight, even in the most extreme conditions.

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The structures of the elliptical parabola are especially designed to reflect thousands of pinpoint rays of light indirectly on the surgical site, thereby eliminating all areas of shadow.

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The "sun light" coating applied to the ellipse and the "dichroic" coating applied to the parabola raise the colour temperature to 4,200°K, generating a white light similar to that of the sun, thereby protecting the dental surgeon's sight and guaranteeing absolute precision of vision.

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To provide maximum depth, Tecno-Gaz lamps have been especially designed to use reflectors with geometry composed of dozens of interconnected ellipsoids which direct the rays to form a cone of light with the power to penetrate to a considerable depth.

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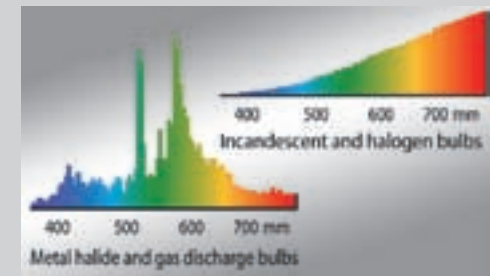
Tecno-Gaz has dedicated particular attention to this aspect, adopting special "RMS97 filters" which absorb and eliminate the infrared rays generated by the halogen bulb, guaranteeing a cold light.

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
Tecno-Gaz lamps use halogen bulbs, because the visible spectrum emission is most similar to that of sun light and this guarantees visual consistency.

# Un sistema innovativo "CONCETTO ELLITTICO" An innovative system "ELLIPTIC CONCEPT"

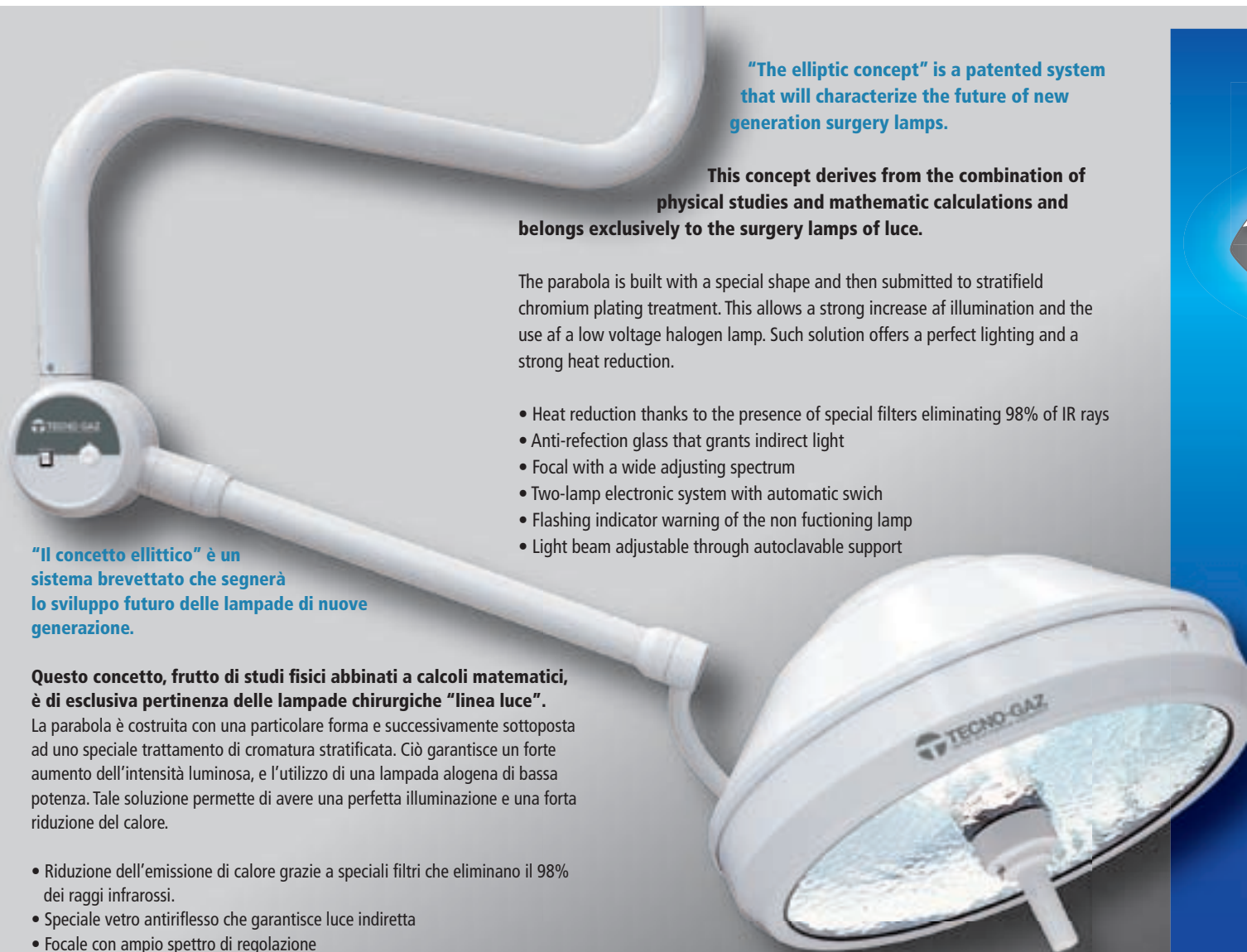**"The elliptic concept" is a patented system that will characterize the future of new generation surgery lamps.**

**This concept derives from the combination of physical studies and mathematic calculations and belongs exclusively to the surgery lamps of luce.**

The parabola is built with a special shape and then submitted to stratified chromium plating treatment. This allows a strong increase of illumination and the use of a low voltage halogen lamp. Such solution offers a perfect lighting and a strong heat reduction.

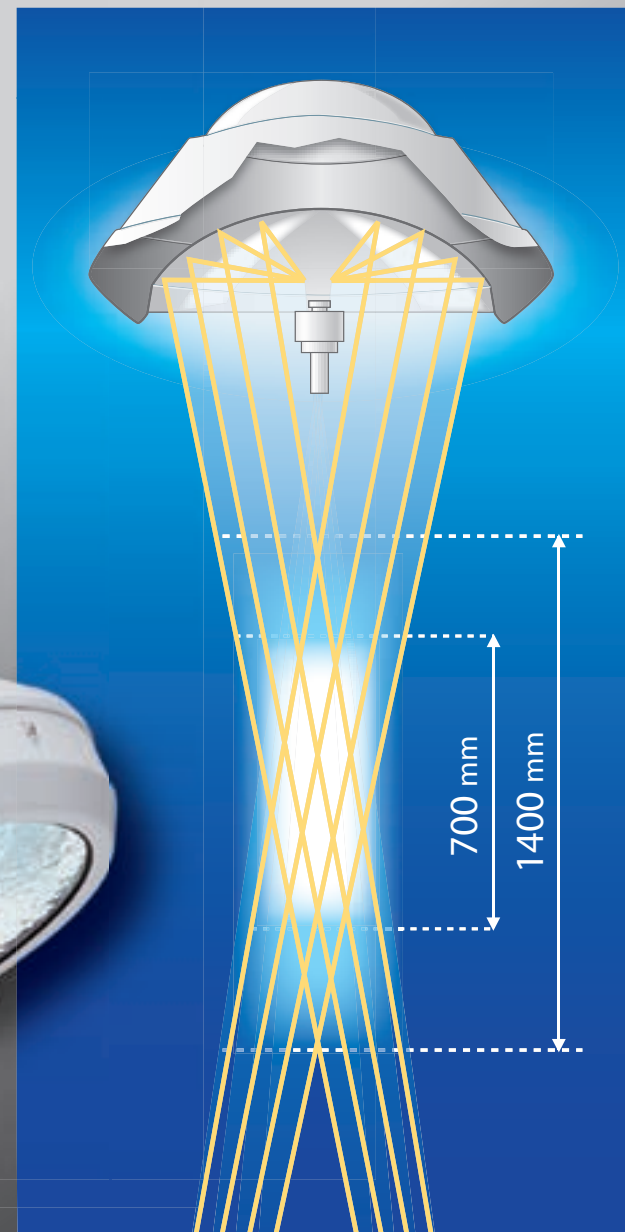
- Heat reduction thanks to the presence of special filters eliminating 98% of IR rays
- Anti-reflection glass that grants indirect light
- Focal with a wide adjusting spectrum
- Two-lamp electronic system with automatic switch
- Flashing indicator warning of the non functioning lamp
- Light beam adjustable through autoclavable support

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# Special features of the LUCE line lamps

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- Tecno-Gaz lamps are always focused at a minimum distance of 60 cm.
- The diameter of the illuminated field may be adjusted manually by turning the knob located at the centre of the reflector. This knob may be easily removed by the operators for sterilization purposes.
- All Tecno-Gaz models are equipped with the lighting intensity control as standard, enabling electronic adjustment via potentiometer from 20.000 Klux up to maximum intensity. An absolutely indispensable option providing the dentist with a multi-purpose lamp suitable for all types of operating activities.
- Tecno-Gaz has dedicated particular attention to the aspects of manoeuvrability and ergonomics during operation. Surgery in an outpatients' context, especially dental surgery, requires a great many adjustments of the arm and, in order to guarantee ease of use, maximum manoeuvrability has been achieved in a structure made entirely of aluminium, which also has the advantage of being extremely light.
- The joints of the articulated arms have bushings to facilitate positioning and guarantee absolute stability.
- Thanks to its lightness, the device is extremely easy to mount on any type of ceiling.

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### → Depth

The rays generated by the lamp are reflected by the elliptical reflector onto the surgical site in pinpoint spots to ensure that the light is constantly focused and three-dimensional. Thanks to this principle, the light is always focused at a minimum distance of 60 cm from the reflector.

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### → Spotlight control

The spotlight can be adjusted manually by turning the knob located at the centre of the reflector. This knob may be easily removed by the operators for sterilization purposes.

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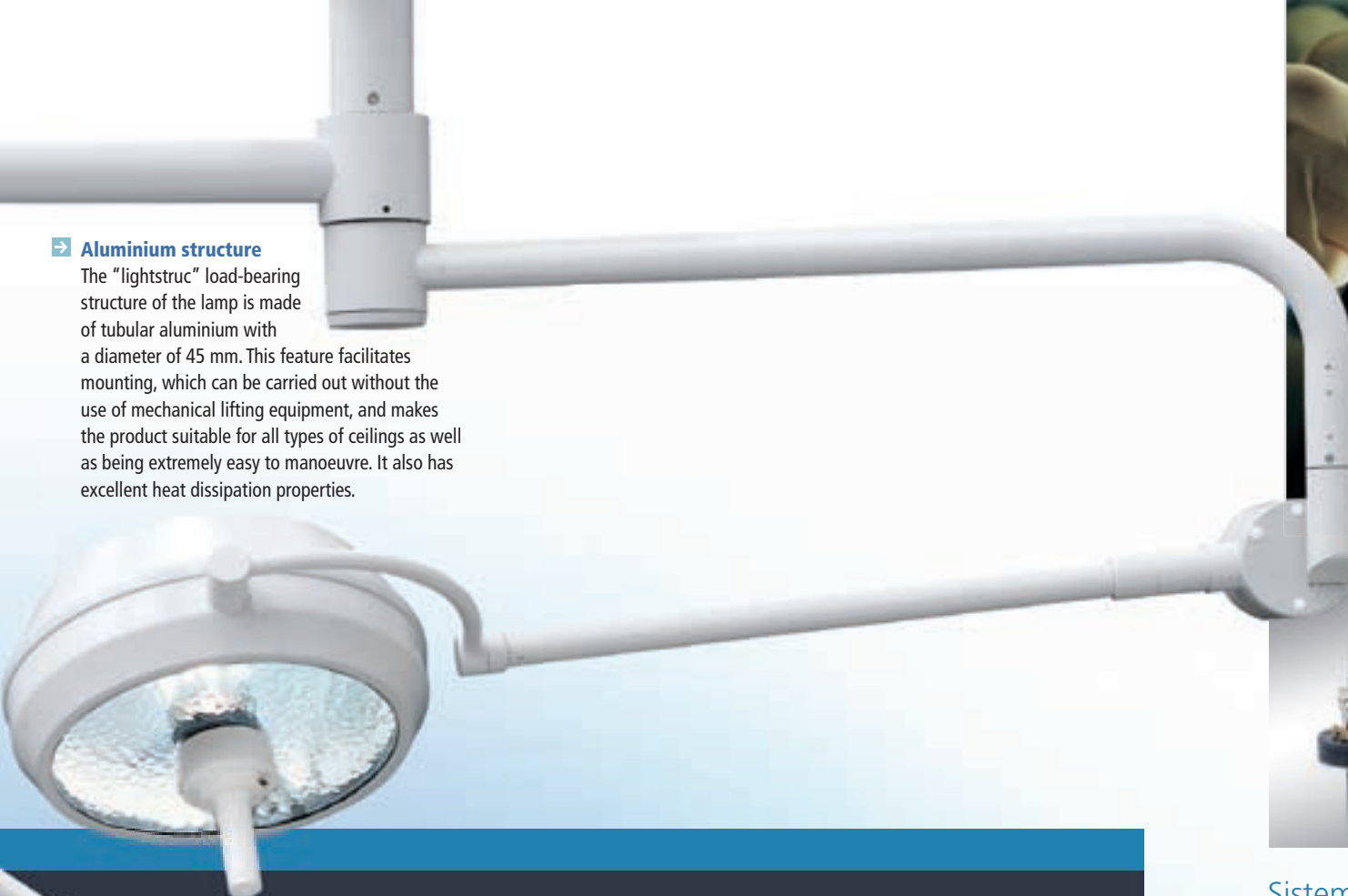
### → Cold light

Special "RMS97 filters" eliminate the infrared rays generated by the halogen bulb, in order to safeguard against dehydration of tissue during surgery and facilitate the dental surgeon's work.

### → Aluminium structure

The "lightstruc" load-bearing structure of the lamp is made of tubular aluminium with a diameter of 45 mm. This feature facilitates mounting, which can be carried out without the use of mechanical lifting equipment, and makes the product suitable for all types of ceilings as well as being extremely easy to manoeuvre. It also has excellent heat dissipation properties.

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### → Elliptical reflector

This is made of aluminium and is composed of dozens of interconnected ellipsoids. The elliptical shape guarantees excellent lighting intensity with a low voltage halogen bulb and the reflected light is always strongly focused. The elliptical reflector has a "mirror" coating to increase the reflection and a "sun light" coating to raise the colour temperature to 4.200 Klux with colour rendering index of 94.

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### Automatic bulb changeover system \*

The lamps are equipped with a block composed of 2 halogen bulbs which are electronically controlled in the event of a failure in the main bulb. The second bulb automatically comes into operation in place of the faulty one. Simultaneously, a LED located on the cover of the lamp starts flashing indicating that the main bulb requires replacing.

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\* standard supply on models D60 and D90, on request on models D40.

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### Lamp replacement signal\*

There is a LED indicator on the cover of the lamp that starts flashing to attract the operator's attention so that he/she proceeds to replace the faulty bulb.

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## 10° rotation

The standing version has been especially designed to be rotated by 10° on a vertical pole, without rotating the base. This technical solution makes it particularly suitable in surgeries where, for logistical reasons, it is difficult to move the base on wheels.

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## Anti-static wheels with brake

The mobile standing version is equipped with four anti-static wheels, two of which with brakes, as standard.

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Stativo / On trolley  
Parete / Wall mounted  
A soffitto / Ceiling

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## Lighting intensity control

Lighting intensity can be electronically regulated thanks to a potentiometer from 20.000 Klux to maximum intensity. The surgeon can set the lighting intensity required for the type of activity to be carried out.

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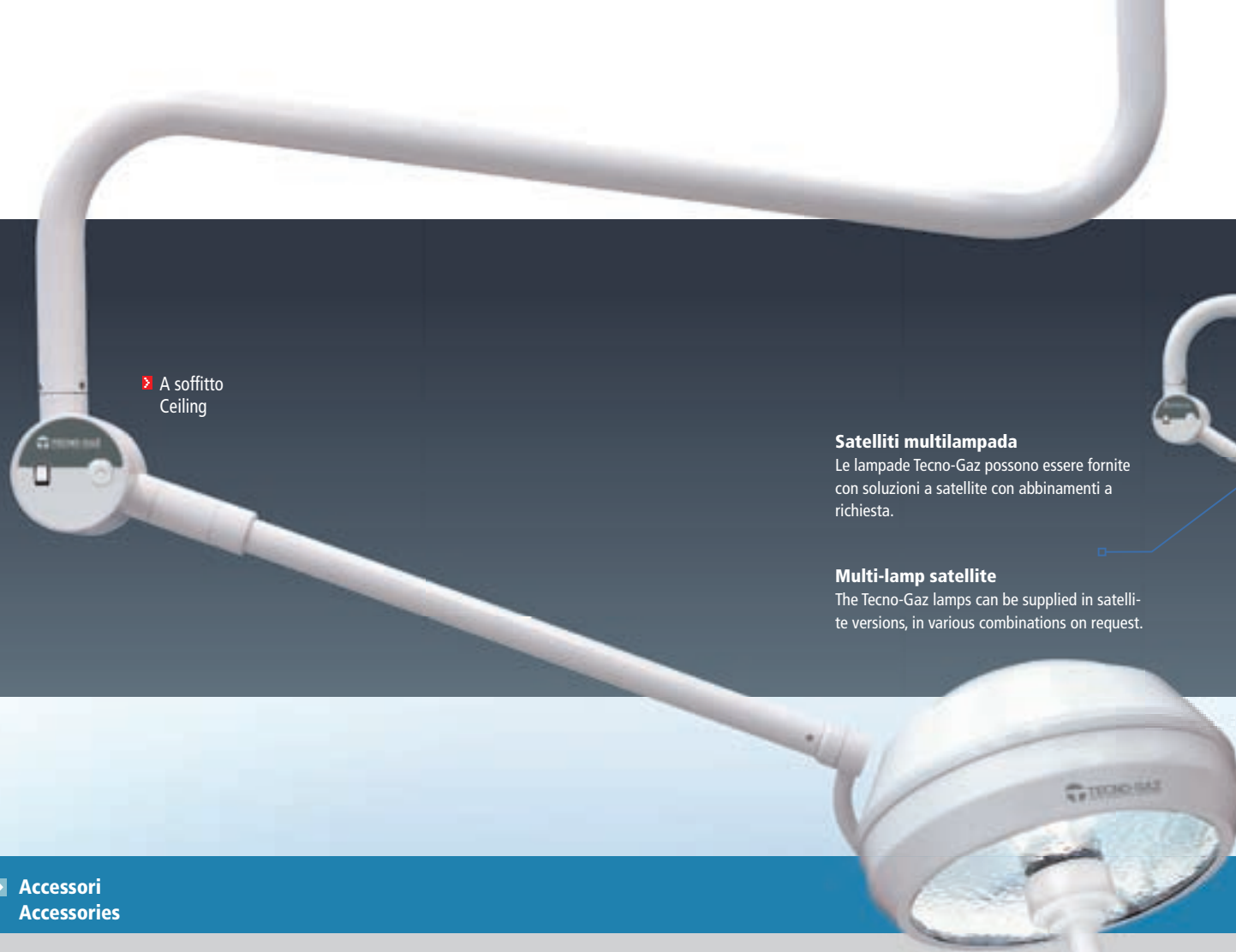
## Operating adjustments

The vertical arm has a yaw of 97° and a stroke of 1455 mm. It is balanced by a compression spring and can be regulated should this be required over time. It is light and its special bushings guarantee stability in all positions.

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### Multi-lamp satellite

The Tecno-Gaz lamps can be supplied in satellite versions, in various combinations on request.

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DI SERIE

### Lighting intensity control

Lighting intensity can be electronically regulated thanks to a potentiometer from 20.000 Klux to maximum intensity.  
STANDARD SUPPLY.

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A RICHIESTA

### Reserve bulb

The lamp can be equipped with a reserve bulb. In the event of a failure in the main bulb, the device instantly switches on the reserve bulb.  
ON REQUEST.

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A RICHIESTA

### Emergency battery

Each Tecno-Gaz lamp can be equipped with an emergency battery, battery charger and electrical circuit.  
ON REQUEST.

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## Operating adjustments

The vertical arm has a yaw of 97° and a stroke of 1455 mm. It is balanced by a compression spring and can be regulated, should this be required over time. It is light and its special bushings guarantee stability in all positions.

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### Reserve bulb

The lamp can be equipped with a reserve bulb. In the event of a failure in the main bulb, the device instantly switches on the reserve bulb.  
STANDARD SUPPLY.

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## Surgical mono-bulb lamp available in ceiling version

Equipped with automatic dual-bulb system with automatic changeover in the event of a failure in the main bulb and intermittent LED indicator on the cover. Spot regulation and lighting intensity control supplied as standard.

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All rights reserved. Variations can be done without notice. Tecno-Gaz spa is to be considered not responsible for damages caused by the lack or the wrongness of the information here mentioned.