Pura • Distiller

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Lengthening the functional life of equipment and instruments

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In the field of sterilization, using distilled or demineralized water is a must for guaranteeing equipment functionality and avoiding problems with the treated materials.

Pure water in fact prevents the formation of stains and rust on sterilized instruments, and protects the autoclave from incrustations that could seriously damage many internal parts, including solenoid valves, membranes pumps and so on.

Over time, scale and hard water residues in the chamber and tanks can cause these materials to deteriorate, leading to the inevitable consequence of costly maintenance.

There are a number of different treatment systems available that provide water at a lower cost than those commonly found on the market, and which have the added advantage of delivering the required product at the right time.

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
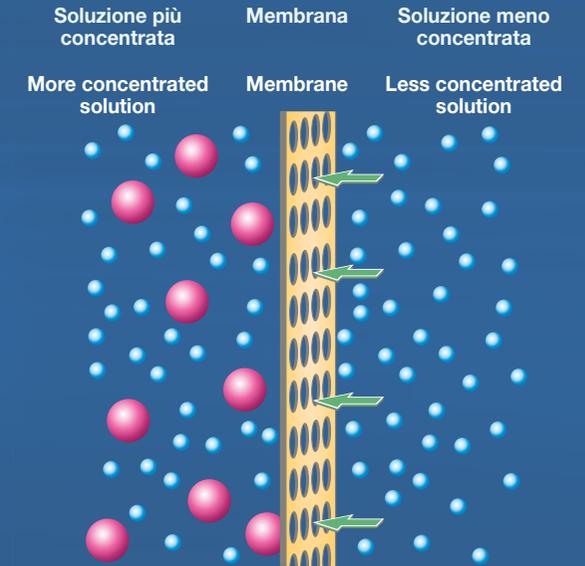
Inverse osmosis is the safest and most widely used water purification system in the world.

In fact, through this system pollutant particles of infinitesimal dimensions are removed, thereby guaranteeing water purity. This is made possible by an osmotic membrane, the real focal point of the system, which has the capability to filter water down to molecular dimensions, endowing it with optimal characteristics as regards both bacteriological and organoleptic purity. **Inverse osmosis is, as the name itself suggests, the inverse process to that which usually occurs in nature.** This procedure is achieved thanks to a force (the pressure of the hydraulic network) which pushes the water with the more concentrated solution, i.e. that "with pollutants" towards a semipermeable osmotic membrane, thereby producing pure water. In this way 99.99% of organic substances and over 90% of inorganic substances are removed.

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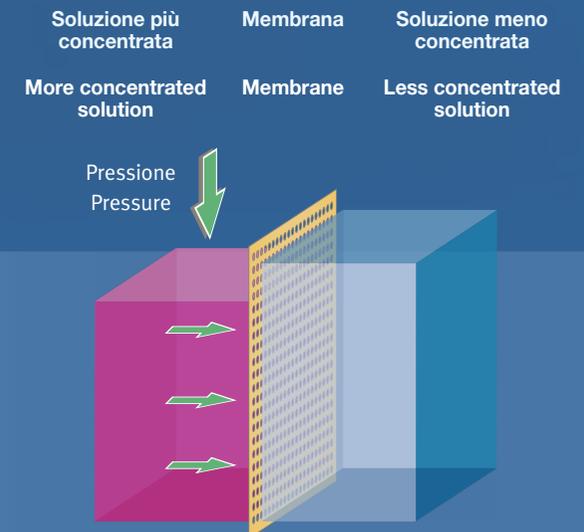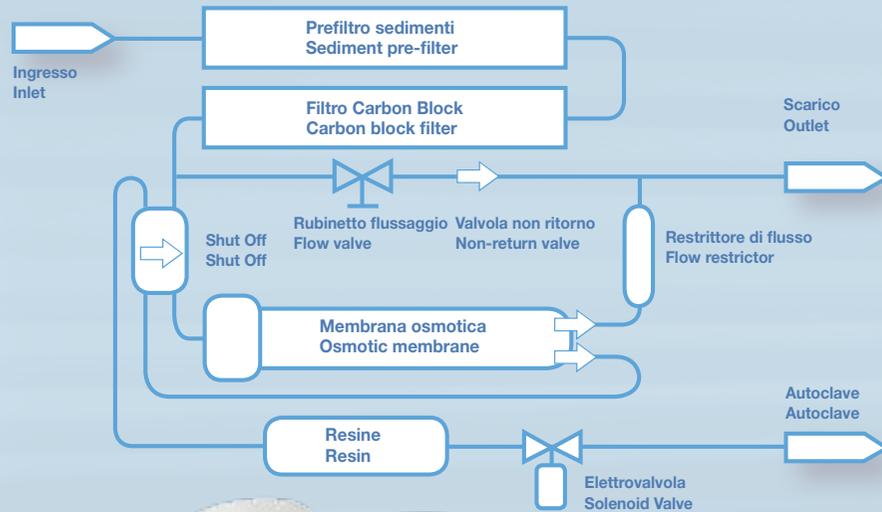
Pura

The Pura model filtering system is an extremely compact inverse osmosis apparatus specifically designed to be hooked up directly to the autoclave, thereby guaranteeing a constant supply of pure water. The system can be installed in a variety of positions, thereby facilitating assembly and positioning in the space available. **The Pura model** has an osmotic membrane and a series of cartridge filters, the average duration of which varies from 600 to 800 litres according to the initial hardness of the water. The cartridges must, therefore, be replaced every 800 cycles or once a year if the autoclave doesn't reach the above mentioned number of cycles. **Filter changing is extremely simple** and can be performed by the operators. **Filters are reasonably priced** so that the cost/litre is extremely cheap. When **the Pura system** is not producing water it is not pressurized, thereby avoiding problems in the event of accidental breaking of fittings or couplings.

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◀ Art.250/S
Distiller

Tecno-Gaz spa presents distiller, a simple, sure and rational system that allows you to have high quality distilled water, thanks to a boiling procedure with forced recondensation. The employment of Distiller will allow you to stop buying and stocking cumbersome water barrels.

Distiller is supplied with accessories, filters and other equipments.


The system

It is enough introducing tap water in the container of the **Distiller** and starting the cycle. The temperature increases and gets to about 100°C. During this phase the chlorine and other waste gases are expelled through an appropriate valve. Water is transformed into steam, which flows into a special coil that cools it and transforms it into distilled water. Before being collected in the special container, the water passes through a filter that eliminates any odours.

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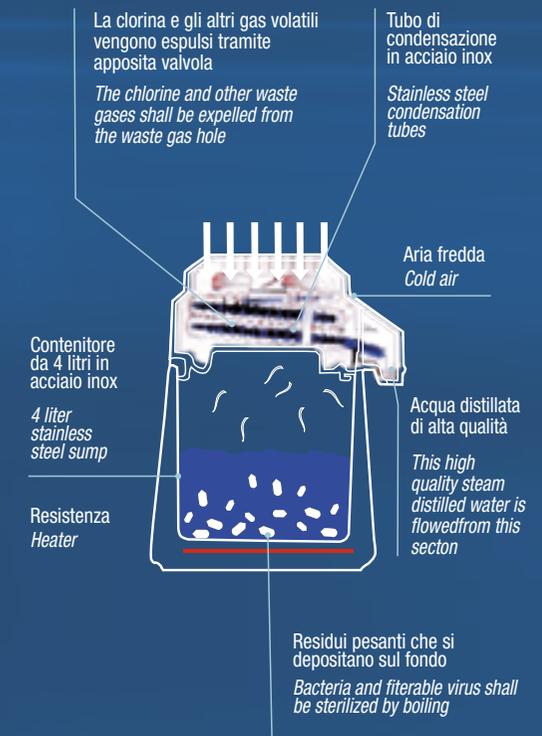All rights reserved. Variations can be done without notice. Tecno-Gaz spa is to be considered not responsible for damages caused by the lack or the wrongness of the information here mentioned.